

# Magnuson Dental Design

## **OCCLUSAL GUARD**

### **Cleaning and Maintenance**

It is important that you keep your appliance clean to maximize its longevity. Upon removal, rinse and brush with a soft toothbrush and toothpaste. You can also soak the appliance in a commercial denture cleaner, but no more than 20 minutes once a week. Store your appliance in the container provided. It is normal for the acrylic to yellow over time.

2901 South Westnedge Avenue · Kalamazoo, MI 49008 · 269-382-4300

[www.MDentalDesign.com](http://www.MDentalDesign.com)